THUASNE

Ortel C1 anatomic

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(en)

Properties:

- Anatomically shaped support foam. (1)
- Precise velcro adjustment. (2)
- Rounded edges for optimum comfort. (3)
- Removable comfortable cover.

Indications:

- Analgesic support for the cervical spine (torticollis, benign sprains, cervical arthritis).

Contraindication:

Do not apply the product in direct contact with broken skin.

Side effects:

No side effects are as yet known for correct use.

Fitting instructions:

- Open the Velcro of the cervical collar.
- Put the cervical collar around the neck. The chin should lie in the area of the gap.
- Close the Velcro in the nape.
- Be careful not to close the strap too tightly.

Precautions:

Follow the advice of the health professional who prescribed or supplied the product. Consult your health professional if discomfort occurs.

Store at room temperature, preferably in its original packaging.

For hygiene and performance reasons, do not re-use the product for another patient.

Washing instructions:

- See product label.
- Do not use detergents, fabric softeners or aggressive products (products containing chlorine)
- Squeeze out excess water.
- Dry away from heat sources.

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Thuasne Benelux B.V.
Thuasne Scandinavia AB
Thuasne Cervitex Ltd
Thuasne UK Ltd
Thuasne (Shanghai) Medical Devices Co., Ltd.