# **ORTEL<sup>®</sup> C2 PLUS**

*Collier cervical semi-rigide*

*Semi-rigid cervical collar*

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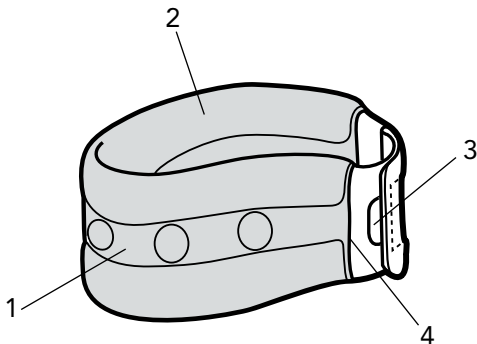
# SEMI-RIGID CERVICAL COLLAR

## Properties :

- Semi-rigid, ventilated light structure. (1)
- Anatomical fit.
- Rounded foam edges for optimum comfort. (2)
- Precise velcro adjustment. (3)
- Removable cover for easy washing. (4)

Heights : 7,5 cm - 9 cm - 11 cm.

## Indications :

Reinforced and analgesic support for the cervical spine (benign sprains, cervical arthritis, torticollis, cervico-brachial neuralgia).

## Contraindication :

Do not apply the product in direct contact with broken skin.

## Fitting instructions :

- Open the Velcro of the cervical collar.
- Put the cervical collar around the neck. The chin should lie in the area of the gap.
- Close the Velcro in the nape.
- Be careful not to close the strap too tightly.

## Precautions :

Follow the advice of the professional who prescribed or supplied the product. Consult your professional if discomfort occurs. Store at room temperature, preferably in its original packaging. For hygiene and performance reasons, do not re-use the product for another patient.

## Washing instructions :

- See product label.
- Do not use detergents, fabric softeners or aggressive products (products containing chlorine)
- Squeeze out excess water.
- Dry away from heat sources.

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