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Description/Destination

Axilla crutch.

This device helps temporarily or permanently mobility impaired individuals to maintain their freedom of movement by providing them with an additional point of support. It can be used inside and outside.

Main components

Tubes: aluminum.

Soft handle and axilla pad: thermoplastic.

Ferrule: PVC.

Indications

Post-operative and post-traumatic support.

Muscle weakness, neuromuscular disorders, stability problems, impaired balance, arthritis of the lower limbs. Degenerative conditions of the lower limbs.

Contraindications

Not suitable for patients weighing > 130 kg.

Do not use in the event of known allergy to any of the components.

Precautions

⚠ Important warnings

Verify the product's integrity before every use.

Do not use the device if it is damaged.

It is recommended that a health care professional supervises the first use.

Follow the advice of the health care professional who prescribed or supplied the product and consult them in case of doubt.

Never press against the armpit for support. Always leave approximately 5 cm between the top of the crutches and the armpits.

Before using this product, check that it is stable, rigid and that all screws are tight.

In case of any visible damage or significant wear of the axilla pad, soft handle or ferrule, contact a specialized dealer or health care professional to have the part replaced.

Do not make any mechanical changes to the device, this could compromise its safety. Only original spare parts and accessories have to be used.

Take care when using the device on slippery surfaces (wet, snow-covered or icy floors or wet tiles, etc.), uneven or unstable surfaces (gravel, sand, leaves, etc.).

Do not leave exposed to direct sunlight for a long period of time.

Undesirable side-effect

- Risk of incorrect use or overuse of the device: muscle fatigue, inflammation or injury. In case of incorrect height adjustment: skin reactions (redness, rashes, irritation, blisters, etc.) or wounds of varying severity, shoulder joint strain, pain in the hand, wrist, forearm or arm (biceps), hematomas and bruises, etc.
- Danger of falling or loss of balance, which can lead to serious injuries.

Any serious incidents concerning related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

Instructions for use

Recommended user height: size S: from 139 to 159 cm - size M: from 159 to 179 cm - size L: from 179 to 199 cm. The height adjustment of a crutch must be done with reference to the height of the patient. Incorrect height adjustment can impair performance and place too much strain on the user.

Crutch height adjustment:

After adjustment, when standing up straight with one hand on the handle, the top of the crutch must be approximately 5 cm from the armpit, with the elbows slightly bent (Fig. A). Support the weight with the hands and not against the armpits.

Adjustment by push-buttions: with a finger on each push-button located on the lower part of the crutch, push both buttons simultaneously while pulling the interior tube in order to adjust the height. Make sure that the crutch is locked in position by pressing it with increasing force on the ground.

Make sure that both crutches have been adjusted to the same height.

Crutch handle position adjustment:

Unscrew the thumb screw, position the handle to the desired height. Make sure that the screw is tightened correctly.

Use:

Never place the crutch too far to the side or too far in front, as this creates a risk of falling.

When in use, grasp the handle of the device completely and firmly.

In case of falling: let the crutch slip to the side so that you do not fall on it.

⚠ Important warnings

Always keep one hand on the handrail when going up or down stairs (Fig. B).

If possible, ask somebody for assistance.

When it is necessary to take stairs without a handrail, only do so with extreme caution.

Only use the device as intended and not for any other use: pulling, lifting or moving objects (Fig. C).

Maintenance and care

Maintenance

If any part of the device becomes detached or loose, contact a specialized dealer or health care professional. If necessary, replacement parts and original accessories are available from specialized dealers for the whole range of Thuasne products.

Care

Clean with a mild cleaning agent and a soft cloth.

Do not clean the handle or the ferrule with greasy substances, this could create a slip hazard.

If necessary, use a mild contact disinfectant or 70° alcohol.

Storage

Store at room temperature.

Disposal

Dispose of in accordance with local regulations.

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