


THUASNE®

en	Sport anti-epicondylitis elbow brace	4
fr	Coudière anti-épicondylite sport	5
de	Sport-Ellenbogenstütze gegen Epicondylitis	6
nl	Anti-epicondylitis elleboogbrace sport	7
it	Gomitiera anti-epicondilite per uso sportivo	9
es	Codera antiepicondilitis para uso deportivo	10
pt	Cotoveleira antiepicondilite para uso desportivo	11
da	Albuebind mod epicondylitis sport	12
fi	Epikondyliittituki urheilukäyttöön	13
sv	Armbågsskydd mot epikondylit sport	15
el	Αθλητική περιαιγκωνίδα κατά της επικονδυλίτιδας	16
cs	Loketní bandáž proti epikondylitidě, sport	17
pl	Opaska zapobiegająca entezopatii stawu łokciowego (łokieć tenisisty) sportowa	18
lv	Sporta elkoņa balsts epikondilīta novēršanai	19
lt	Alkūnės įtvaras epikondilito prevencijai sportuojant	21
et	Epikondüliidi küünarnukitugi sportimiseks	22
sl	Komolčna opornica proti epikondilitisu šport	23
sk	Športová podpora lakťa proti epikondylitíde	24
hu	Teniszkönyök elleni könyökrögzítő sportoláshoz	25
bg	Налакътник срещу епикондилит за спорт	26
ro	Cotieră antiepicondilită sport	28
hr	Sportski steznik protiv epikondilitisa	29

SPORT

Instructions for use - Instructions d'utilisation - Gebrauchsanweisung - Gebruiksaanwijzingen - Istruzioni per l'uso - Instrucciones de uso - Instruções de utilização - Anvisninger vedrørende brug - Käyttöohjeet - Råd vid användning - Οδηγίες χρήσης - Pokyny k používání - Szczegóły użytkowania - Lietošanas instrukcija - Naudojimo instrukcijos - Kasutusjuhised - Navodila za uporabo - Pokyny na používanie - Használati utasítás - Инструкции за употреба - Instrucțiuni de utilizare - Upute za uporabu

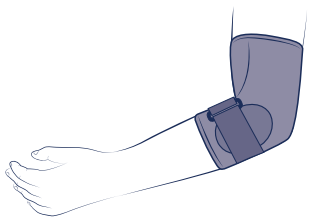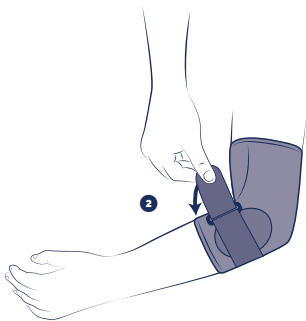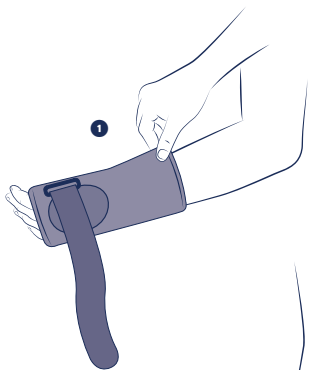cm

S

22 - 24

M

25 - 27

L

28 - 30

XL

31 - 33

en	Forearm circumference
fr	Tour de l'avant-bras
de	Umfang des Unterarms
nl	Omtrek van de voorarm
it	Circonferenza dell'avambraccio
es	Circunferencia del antebrazo
pt	Perímetro do antebraço
da	Omkreds af underarm
fi	Yläkäsivarren ympärysmitta
sv	Runt underarmen
el	Περίμετρος του αντιβραχίου
cs	Pro předloktí
pl	Opaska na przedramię
lv	Apkārt apakšdelmam
lt	Dilbio apimtis
et	Küünarvarre ümbermõõt
sl	Obseg podlakti
sk	Obvod predlaktia
hu	Alkar kerülete
bg	Обиколка на предмишницата
ro	Circumferința antebrațului
hr	Opseg podlaktice

en

SPORT ANTI-EPICONDYLITIS ELBOW BRACE

Description/Destination

This device is an elbow brace providing joint support and stabilization.

This device can be used when resuming sports activities.

The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table.

Composition

Textile components: synthetic rubber (SBR) - polyamide - elastane.

Strap: polyester - polyoxymethylene.

Insert: silicone.

Properties/Mode of action

Even compression on the joint.

Improved proprioception.

Removable tightening strap.

Indications

Lateral or medial epicondylitis (tennis elbow/golfer's elbow).

Prevention of elbow tendinopathy recurrence.

Contraindications

Do not use the product if the diagnosis has not been confirmed.

Do not apply the product in direct contact with broken skin.

Do not use in the event of known allergy to any of the components.

Precautions

Verify the product's integrity before every use.

Do not use the device if it is damaged.

Choose the appropriate size to fit the patient, referring to the size chart.

In the event of discomfort, significant hindrance, pain, variation in limb volume, abnormal sensations or change in colour of the extremities, remove the device and consult a healthcare professional.

For hygiene and performance reasons, do not re-use the product for another patient.

Do not use the device in case of application of certain products on the skin (creams, ointments, oils, gels, patches...).

It is recommended to adequately tighten the device to achieve support without restricting blood circulation.

Undesirable side-effect

This device can cause skin reactions (redness, itching, burns, blisters...) or wounds of various degrees of severity.

Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

Instructions for use/Application

Slip on the elbow brace, taking care to position the inserts on the forearm. ❶

Fasten the strap on the elbow brace using the self-fastener. ❷

Do not tighten excessively to avoid any tourniquet effect.

Garment care

Product can be washed in accordance with the conditions shown on this instruction leaflet and on the label. Do not use detergents, fabric softeners or aggressive products (products containing chlorine, etc.).

Squeeze out excess water. Dry away from any direct heat source (radiator, sun, etc.).

Storage

Store at room temperature, preferably in the original packaging.

Disposal

Dispose of in accordance with local regulations.

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©Thuasne - 2051801 (2024-02)
THUASNE UK Ltd
Thuasne SAS - SIREN/RCS Nanterre 542 091 186