# THUASNE

# SPORT

<b>fr</b>	<b>Genouillère tricotée de maintien renforcée Thuasne Sport</b> .....	3
<b>en</b>	<b>Thuasne Sport reinforced knitted knee support</b> .....	4
<b>de</b>	<b>Verstärkte gestrickte Thuasne Sport-Kniebandage</b> ...	5
<b>nl</b>	<b>Thuasne Sport verstevigde gebreide steunbrace voor de knie</b> .....	7
<b>it</b>	<b>Ginocchiera in maglina di sostegno rinforzata Thuasne Sport</b> .....	8
<b>es</b>	<b>Rodillera de sujeción reforzada de punto Thuasne Sport</b> .....	10
<b>pt</b>	<b>Joelheira tricotada de suporte reforçada Thuasne Sport</b> .....	11
<b>da</b>	<b>Strikket forstærket knæstøttebind Thuasne Sport</b> ...	13
<b>fi</b>	<b>Vahvistettu kudottu Thuasne Sport -polvituki</b> .....	14
<b>sv</b>	<b>Thuasne Sport stickat förstärkt knästöd</b> .....	15
<b>el</b>	<b>Πλεκτή ενισχυμένη επιγονατίδα συγκράτησης Thuasne Sport</b> .....	17
<b>cs</b>	<b>Pletená vyztužená kolenní ortéza Thuasne Sport</b> .....	18
<b>pl</b>	<b>Podtrzymująca, dzianinowa orteza kolanowa, wzmocniona Thuasne Sport</b> .....	20
<b>lv</b>	<b>Trikotāžas ceļa atbalsts ar pastiprinājumu Thuasne Sport</b> .....	21
<b>lt</b>	<b>Megztas sutvirtintas kelio palaikomasis įtvaras „Thuasne Sport“</b> .....	22
<b>et</b>	<b>Kootud kangast Thuasne Sport tugevdatud põlveortoos</b> ...24	
<b>sl</b>	<b>Ojačana pletena opornica za koleno Thuasne Sport</b> ...25	
<b>sk</b>	<b>Vystužená štrikovaná kolenná ortéza Thuasne Sport</b> .....26	
<b>hu</b>	<b>Thuasne Sport megerősített kötött támasztó térdmerevítő</b> .....	28
<b>bg</b>	<b>Подсилена поддържаща плетена накованка Thuasne Sport</b> .....	29
<b>ro</b>	<b>Genunchieră tricotată de susținere ranforsată Thuasne Sport</b> .....	31
<b>ru</b>	<b>Поддерживающий трикотажный наколенник Thuasne Sport с усиленной защитой</b> .....	32
<b>hr</b>	<b>Pojačana pletena potpora za koljeno Thuasne Sport</b> .....34	
<b>zh</b>	<b>Thuasne Sport 针织加强稳定护膝</b> .....	35
<b>ar</b>	<b>Thuasne Sport داعم الركبة المحبوك المعزز</b> .....	39

 cm	<b>S</b> 32 - 34	<b>M</b> 35 - 37	<b>L</b> 38 - 41	<b>XL</b> 42 - 45	<b>XXL</b> 46 - 49
--	------------------	------------------	------------------	-------------------	--------------------

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---

#### **en**

### **THUASNE SPORT REINFORCED KNITTED KNEE SUPPORT**

#### **Description/purpose**

The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table.

Bilateral product.

Available in 5 sizes.

The device is composed of:

- an anatomically-shaped knitting integrating comfortable areas over the patella and at the back of the knee.
- flexible side boning,
- pull tab for easier fitting.

#### **Composition**

Textile components: polyamide - elasthan - polyester - elastodiene - viscosse - naturel latex - silicone

Rigid components: polyurethane - polyethylene - steel

#### **Properties/Mode of action**

Elastic knit ensuring proprioception.

#### **Indications**

Knee pain and/or swelling management.

Joint instability/laxity.

Sport activities resumption.

#### **Contraindications**

Do not use in the event of known allergy to any of the components.

Do not apply the product in direct contact with broken skin.

History of venous or lymphatic disorders.

Do not use in the event of major venous thromboembolic history without thromboprophylaxis.

#### **Precautions**

Verify the product's integrity before every use.

Do not use the product if it is damaged.

Choose the appropriate size to fit the patient, referring to the size chart. For hygiene and performance reasons, do not re-use the product for another patient.

Consult a healthcare professional in case of uncertain diagnosis.

In the event of discomfort, significant hindrance, pain, variation in limb volume, abnormal sensations or change in colour of the extremities, remove the device and consult a healthcare professional.

In the event of any modification in the product's performance, remove it and consult a healthcare professional.

Contains natural latex.

In case of doubt before any sports activity, consult a healthcare professional.

Do not wear the product in a medical imaging machine.

Do not use the device in case of application of certain products on the skin (creams, ointments, oils, gels, patches...).

#### **Undesirable side-effects**

This device can cause skin reactions (redness, itching, burns, blisters, etc.) or wounds of various degrees of severity.

Any serious incidents related to the product should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

#### **Instructions for use/Application**

It is recommended to wear the device directly on the skin, unless contraindicated.

Slip the knee brace on the leg, making sure that the kneecap is properly positioned in the flexible knitted zone provided for this purpose.

Thuasne Logo must be on the upper part, centered and focused on the patella.

#### **Care**

Hand washing. Do not use detergents, fabric softeners or aggressive products (products containing chlorine). Squeeze out excess water.

Dry away from any direct heat source (radiator, sun, etc.). If the device is exposed to seawater or chlorinated water, make sure to rinse it in clear water and dry it.

#### **Storage**

Store at room temperature, preferably in the original packaging.

#### **Disposal**

Dispose of in accordance with local regulations.

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©Thuasne - 2035101 (2022-08)
Thuasne SAS – SIREN/RCS Nanterre 542 091 186