THUASNE

SPORT

fr	Genouillère baleinée Thuasne Sport	3
en	Thuasne Sport reinforced knee support	4
de	Thuasne Sport-Kniebandage mit Stützstäben	5
nl	Thuasne Sport kniebrace met baleinen	7
it	Ginocchiera dotata di stecche Thuasne Sport	8
es	Rodillera con ballenas Thuasne Sport	10
pt	Joelheira almofadada Thuasne Sport	11
da	Knæbind med afstivere Thuasne Sport	13
fi	Vahvikkeellinen Thuasne Sport -polvituki	14
sv	Knästöd med skenor Thuasne Sport	15
el	Επιγονατίδα με μπανέλες Thuasne Sport	17
cs	Kolenní ortéza s kosticemi Thuasne Sport	18
pl	Orteza kolanowa z fiszbinami Thuasne Sport	19
lv	Stingrā ceļa ortoze Thuasne Sport	21
lt	Kelio įtvaras su plokštelėmis „Thuasne Sport“	22
et	Thuasne Sport küljetugevdusega põlvekaitse	24
sl	Opornica za koleno z ojačitvami Thuasne Sport	25
sk	Kolenná ortéza s bočnými výstužami Thuasne Sport	26
hu	Thuasne Sport térdrögzítő	28
bg	Наколенка с банели Thuasne Sport	29
ro	Genunchieră cu balene Thuasne Sport	30
ru	Наколенник с ребрами жесткости Thuasne Sport ..	32
hr	Thuasne Sport térdrögzítő	33
zh	Thuasne Sport带支撑条的护膝	35
ar	Thuasne Sport داعم ركبة مزود بشرائح	37

 cm	S 31 - 33	M 34 - 36	L 37 - 39	XL 40 - 42	XXL 43 - 45
--	---------------------	---------------------	---------------------	----------------------	-----------------------

fr	Circonférence de genou en légère flexion
en	Knee circumference in slight flexion
de	Knieumfang bei leichter Beugung
nl	Omvang van de licht geplooidie knie
it	Circonferenza del ginocchio in leggera flessione
es	Circunferencia de la rodilla con ligera flexión
pt	Circunferência do joelho em ligeira flexão
da	Knæets omkreds i let bøjning
fi	Polven ympärys hieman koukistettuna
sv	Omkrets runt knäet i lätt böjning
el	Περιφέρεια γόνατος σε ελαφρά κάμψη
cs	Obvod kolene v mírném ohybu
pl	Obwód kolana przy lekkim zgięciu
lv	Celgala apkārtmērs nelielā saliekumā
lt	Šiek tiek sulenкто kelio apimtis
et	Kergelt pinguldatult põlve ümber
sl	Obseg kolena pri rahlem upogibu
sk	Obvod kolena s miernym ohybom
hu	Térd kerülete enyhe hajlításban
bg	Обиколка на коляното при лека флексия
ro	Circumferința genunchiului în flexiune ușoară
ru	Обхват колена в состоянии небольшого сгибания
hr	Obuhvaća koljeno u blagoj fleksiji
zh	轻度弯曲的膝关节周长
ar	محيط الركبة في حالة الثني الخفيف

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en

THUASNE SPORT REINFORCED KNEE SUPPORT

Description/purpose

The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table.

Bilateral product.

Available in 5 sizes.

The device is composed of:

- flexible side boning,
- a synthetic rubber (chloroprene) with an opening in the kneecap.

Composition

Textile components: synthetic rubber (chloroprene) - polyamide - elasthan - polyester - Natural rubber

Rigid component: steel

Properties/Mode of action

The double stays allow a knee joint support.

Limitation of the heat loss by the synthetic rubber (Chloroprene).

Indications

Knee pain and/or swelling management.

Joint instability/laxity.

Sport activities resumption.

Contraindications

Do not use in the event of known allergy to any of the components.

Do not apply the product in direct contact with broken skin.

History of venous or lymphatic disorders.

Do not use in the event of major venous thromboembolic history without thromboprophylaxis.

Precautions

Verify the product's integrity before every use.

Do not use the product if it is damaged.

Choose the appropriate size to fit the patient, referring to the size chart. For hygiene and performance reasons, do not re-use the product for another patient.

Consult a healthcare professional in case of uncertain diagnosis.

In the event of discomfort, significant hindrance, pain, variation in limb volume, abnormal sensations or change in colour of the extremities, remove the device and consult a healthcare professional.

In the event of any modification in the product's performance, remove it and consult a healthcare professional.

Contains natural latex.

In case of doubt before any sports activity, consult a healthcare professional.

Do not wear the product in a medical imaging machine.

Do not use the device in case of application of certain products on the skin (creams, ointments, oils, gels, patches...).

Undesirable side-effects

This device can cause skin reactions (redness, itching, burns, blisters, etc.) or wounds of various degrees of severity.

Any serious incidents related to the product should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

Instructions for use/Application

It is recommended to wear the device directly on the skin, unless contraindicated.

Pull the knee support.

Centre the front opening over the kneecap.

Thuasne Logo must be on the upper part, centered and focused on the patella.

Care

Hand washing. Do not use detergents, fabric softeners or aggressive products (products containing chlorine). Squeeze out excess water. Dry away from any direct heat source (radiator, sun, etc.). If the device is exposed to seawater or chlorinated water, make sure to rinse it in clear water and dry it.

Storage

Store at room temperature, preferably in the original packaging.

Disposal

Dispose of in accordance with local regulations.

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©Thuasne - 2035201 (2022-08)
THUASNE UK Ltd
Thuasne SAS – SIREN/RCS Nanterre 542 091 186