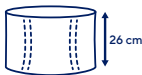
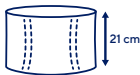
THUASNE

LombaSkin

| | | |
|----|---|----|
| fr | Ceinture de soutien lombaire effet "seconde peau" | 6 |
| en | "Second skin" lumbar support belt | 7 |
| de | LWS-Stabilisierungsothese mit „Zweite-Haut-Effekt“ | 7 |
| nl | Compacte rugbrace voor de onderrug, een "tweede huid" | 8 |
| it | Cintura di sostegno lombare effetto "seconda pelle" | 9 |
| es | Faja de sujeción lumbar con efecto de "segunda piel" | 10 |
| pt | Cinta de suporte lombar efeito "segunda pele" | 11 |
| da | Lændestøttebælte med tætsluttende tøj-effekt | 12 |
| fi | Lannerangan tukivö - "kevyt ja miellyttävä käyttää" | 13 |
| sv | Ländryggsstöd med känslan av "en andra hud" | 14 |
| el | Ζώνη οσφυϊκής υποστήριξης με αίσθηση «δεύτερου δέρματος» | 15 |
| cs | Kompaktní bederní podpurný pás | 16 |
| pl | Orteza lędźwiowo-krzyżowa z efektem „drugiej skóry” | 17 |
| lv | Kompakta jostasvietas josta ar „otrās ādas” efektu | 18 |
| it | Palaikomasis juosmens diržas su „antrosios odos” poveikiu | 19 |
| et | Nimmeiirkonna tugivöö efektiga „teine nahakiht” | 20 |
| sl | Oporni križni pas z občutkom »druge kože« | 21 |
| sk | Podporný drienkový pás s efektom „druhej kože” | 22 |
| hu | „Második bőr” hatású ágyéki fűző | 22 |
| bg | Лумбален колан ефект „втора кожа” | 23 |
| ro | Centură de susținere lombară cu efect de „a doua piele” | 24 |
| ru | Опорный поясничный корсет с эффектом «второй кожи» | 25 |
| hr | Lumbalni potporni pojas s učinkom „druge kože” | 27 |
| zh | “肤质” 效果支撑带 | 27 |
| ar | حزام لدعم أسفل الظهر بتأثير "الجلد الثاني" | 29 |





| | | |
|--|---|--------------|
|  | 1 | 60 - 80 cm |
| | 2 | 75 - 95 cm |
| | 3 | 90 - 110 cm |
| | 4 | 105 - 125 cm |
| | 5 | 120 - 140 cm |

| | | |
|-----------|----------------------|---------------------------|
| fr | Tour de taille | Hauteurs : 21 cm - 26 cm |
| en | Waist measurement | Heights: 21 cm - 26 cm |
| de | Taillenumfang | Höhen: 21 cm - 26 cm |
| nl | Tailleomvang | Hoogtes: 21 cm - 26 cm |
| it | Circonferenza vita | Altezze: 21 cm - 26 cm |
| es | Contorno de cintura | Alturas: 21 cm - 26 cm |
| pt | Perímetro de cintura | Alturas: 21 cm - 26 cm |
| da | Taljemål | Højde: 21 cm - 26 cm |
| fi | Vyötärönympärys | Korkeudet: 21 cm - 26 cm |
| sv | Midjemått | Höjd: 21 cm - 26 cm |
| el | Περίμετρος μέσης | Ύψη: 21 εκ. - 26 εκ |
| cs | Obvod pasu | Výšky: 21 cm - 26 cm |
| pl | Obwód w talii | Wysokości: 21 cm - 26 cm |
| lv | Vidukļa apkārtmērs | Augstumi: 21 cm - 26 cm |
| lt | Juosmens apimtis | Aukštis: 21 cm - 26 cm |
| et | Vööübermõõt | Kõrgused: 21 cm - 26 cm |
| sl | Obseg pasu | Višina: 21 cm - 26 cm |
| sk | Obvod pása | Výšky: 21 cm - 26 cm |
| hu | Derékbőség | Magasság: 21 cm - 26 cm |
| bg | Обиколка на талията | Височини: 21 cm - 26 cm |
| ro | Circumferința taliei | Înălțimi: 21 cm - 26 cm |
| ru | Обхват талии | Высота: 21 см - 26 см |
| hr | Opseg struka | Visine: 21 cm - 26 cm |
| zh | 腰围 | 高度 : 21 cm-26 cm |
| ar | محيط الخصر | الارتفاعات: 21 سم - 26 سم |

| | | | |
|-----------|--------------|----------------|--------------------------------|
| fr | Compression | Maintien | Education posturale |
| en | Compression | Support | Postural education |
| de | Kompression | Halt | Erlernen der Körperhaltung |
| nl | Compressie | Fixatie | Voorlichting over houding |
| it | Compressione | Sostegno | Educazione posturale |
| es | Compresión | Sujeción | Educación postural |
| pt | Compressão | Manutenção | Educação postural |
| da | Kompression | Støtte | Postural genoptræning |
| fi | Kompressio | Tuki | Asentokuntoutus |
| sv | Kompression | Stöd | Postural träning |
| el | Συμπίεση | Συγκράτηση | Σωστή στάση σώματος |
| cs | Komprese | Stabilizace | Správné držení těla |
| pl | Kompresja | Podtrzymywanie | Edukacja dotycząca postawy |
| lv | Kompresija | Atbalsts | Stājas korekcija |
| lt | Kompresija | Palaikymas | Laikysenos taisymas |
| et | Kompressioon | Hooldus | Rühitreening |
| sl | Kompresija | Oprajem | Učenje telesne drže |
| sk | Kompresia | Podpora | Cvičenie posturálnych svalov |
| hu | Kompresszió | Támasz | Tartásjavító oktatás |
| bg | Компресия | Поддръжка | Приучаване към правилна стойка |
| ro | Compresie | Susținere | Educație posturală |
| ru | Компрессия | Стабилизация | Исправление осанки |
| hr | Kompresija | Potpورا | Učenje o držanju |
| zh | 压紧 | 稳定 | 姿势联系 |
| ar | | الضغط | الدعم |
| | | | تدريب وضعي |

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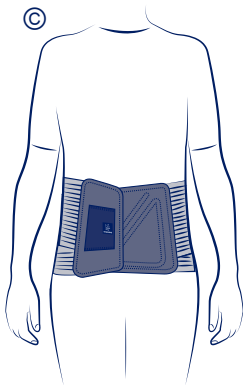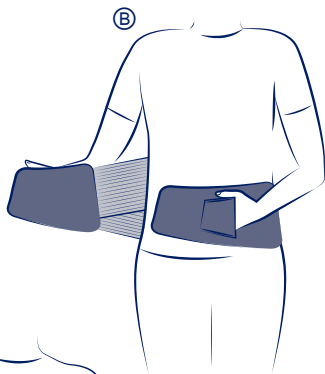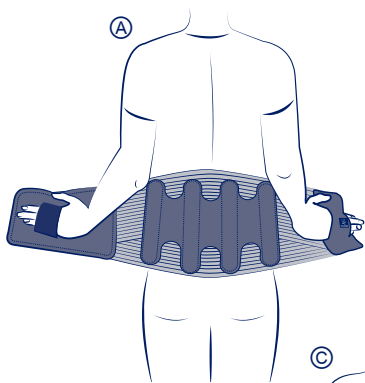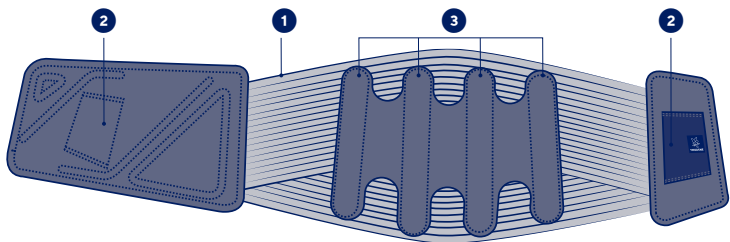en "SECOND SKIN"[™] LUMBAR SUPPORT BELT

The LombaSkin belt is breathable thanks to its CombiteX fabric. The edges of the fabric are seamless to give it greater flexibility for wearing next to the skin, with a "second skin" effect*.

Description/Destination

The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table.

Composition

Aerated and breathable CombiteX fabric. ❶

Abdominal panels with self-fastening surface and finger-loops. ❷

Conformable dorsal stays ❸

Textile components: polyamide - polyurethane - polyester - elastan.

Non-textile components: brushed tempered steel - polyoxymethylene - polypropylene.

Properties/Mode of action

The device relieves low back pain, supporting the lumbar spine: compartmental lumbar unloading is therefore achieved via increased intra-abdominal pressure.

Indications

Non-specific low back pain (acute, subacute and chronic phases).

Spinal stenosis.

Contraindications

Do not apply the product in direct contact with broken skin.

Do not use in the event of known allergy to any of the components.

Do not use on pregnant women.

Do not use in the event of a hiatus hernia.

Do not use in the event of bone cancer with spinal metastases.

Do not use in the event of circulatory, pulmonary or cardiovascular problems in patients for whom an increase in arterial blood pressure is not recommended.

Precautions

Verify the product's integrity before every use.

Do not use the device if it is damaged.

Choose the appropriate size to fit the patient, referring to the size chart.

Strictly comply with your healthcare professional's prescription and recommendations for use.

It is recommended that a healthcare professional adjust the stays to the shape of the patient's back before the belt is worn for the first time.

Do not wear the product in a medical imaging machine.

Do not wear the product while sleeping.

Consult a health professional if discomfort or pain occurs.

The use of a medical device by a child, should be done under the supervision of an adult or a healthcare professional.

Undesirable side-effect

This device can cause skin reactions (redness, itching, irritations...).

Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

Instructions for use/Application

Position the belt around the waist, placing your hands in the finger loops located on the abdominal panels. ❹

The belt must be centred on the spine: the dorsal stays should be positioned on either side of the spine. The bottom of the belt must stop at the top of the crease between the buttocks.

The Thuasne logo indicates which way up the belt should be put on.

Pull the finger loops on the abdominal panels simultaneously and evenly. ❺

Fasten the belt, right side over left side, adjusting to the desired compression. ❻

The level of compression can be adjusted as required throughout the day.

Care

Close the self-fastening tabs before washing. Machine washable at 30°C (delicate programme).

Do not machine wash the device more than 10 times. Do not dry clean. Do not use detergents, fabric softeners or aggressive products (products containing chlorine). Do not tumble-dry. Do not iron. Squeeze out excess water. Dry away from any direct heat source (radiator, sun, etc.). Dry flat.

Storage

Store at room temperature, preferably in the original packaging.

Disposal

Dispose of in accordance with local regulations.

* OC1806 In-house study report, 2019.

This medical device is a regulated health product with CE marking

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©Thuasne - 2124002 (2021-03)