THUASNE

Ligacast®

fr	Attelle stabilisatrice de cheville.....	4
en	Stabilising ankle brace.....	5
de	Orthese zur Stabilisierung des Sprunggelenks.....	7
nl	Stabilisatiebrace voor de enkel.....	8
it	Tutore stabilizzatore per caviglia.....	10
es	Férula estabilizadora de tobillo.....	11
pt	Tala de estabilizadora do tornozelo.....	13
da	Stabiliserende ankelskinne.....	15
fi	Stabiloiva nilkkatuki.....	16
sv	Stabiliseringsskena för ankeln.....	17
el	Νάρθηκας σταθεροποίησης ποδοκνημικής.....	19
cs	Stabilizační ortéza kotníku.....	20
pl	Orteza do stabilizacji kostki.....	22
lv	Potītes stabilizēšanas saite.....	23
lt	Stabilizuojantis kulkšnies įtvaras.....	25
et	Hüppeliigese fikseerimise lahas.....	26
sl	Opornica za stabilizacijo gležnja.....	28
sk	Stabilizačná členková dlaha.....	29
hu	Stabilizáló bokarögzítő.....	31
bg	Стабилизираща шина за глезен.....	32
ro	Atelă stabilizatoare pentru gleznă.....	34
ru	Стабилизирующий ортез для лодыжки.....	35
hr	Udlaga za stabilizaciju gležnja.....	37
zh	踝部稳定夹板.....	39
ar	جبيرة تثبيت الكاحل.....	41


(A)


(B)

(C)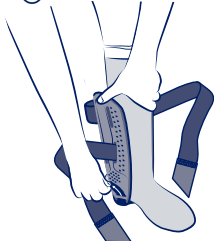

(D)

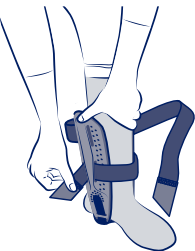
(E)

(F)

(G)

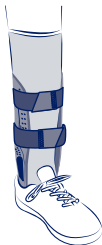	0	< 140 cm
	1	> 140 cm

fr	Taille du patient
en	Patient height
de	Größe des Patienten
nl	Maat van de patiënt
it	Altezza del paziente
es	Altura del paciente
pt	Tamanho do paciente
da	Patientens højde
fi	Potilaan koko
sv	Patientens storlek
el	Μέγεθος ασθενούς
cs	Výška pacienta
pl	Rozmiar pacjenta
lv	Pacienta augums
lt	Paciento dydis
et	Patsiendi suurus
sl	Velikost pacienta
sk	Veľkosť pacienta
hu	A beteg mérete
bg	Размер на пациента
ro	Talia pacientului
ru	Размер пациента
hr	Veličina pacijenta
zh	患者身高
ar	مقاس المريض

fr	Stabilisation
en	Stabilisation
de	Stabilisierung
nl	Stabilisatie
it	Stabilizzazione
es	Estabilización
pt	Estabilização
da	Stabilisering
fi	Stabiloi
sv	Stabilisering
el	Σταθεροποίηση
cs	Stabilizace
pl	Stabilizacja
lv	Stabilizēšana
lt	Stabilizavimas
et	Stabiliseerimine
sl	Stabiliziranje
sk	Stabilizácia
hu	Stabilizálás
bg	Стабилизация
ro	Stabilizare
ru	Стабилизация
hr	Stabilizacija
zh	稳定
ar	الثبيت

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STABILISING ANKLE BRACE

Description/Destination

This device is a ankle brace providing joint stabilisation.

The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table.

Composition

Rigid components: polypropylene - polyurethane foam.

Textile components: polyester - polyamide.

Properties/Mode of action

Lateral rigid plastic shells help stabilise the joint.

Precise fit thanks to adjustable straps.

Heel loop with width adjustment.

Removable foam pads with extra protective layer on the malleoli.

Indications

Moderate to severe ankle sprain.

Chronic ankle instability.

Contraindications

Do not use the product if the diagnosis has not been confirmed.

Do not use in the event of known allergy to any of the components.

Do not apply the product in direct contact with broken skin.

History of venous or lymphatic disorders.

Precautions

Verify the product's integrity before every use.

Do not use the device if it is damaged.

Choose the appropriate size to fit the patient, referring to the size chart.
It is recommended that a healthcare professional supervises the first application.
Strictly comply with your healthcare professional's prescription and recommendations for use.
Check the condition of the affected limb daily (with particular attention for patients with sensory deficit).
In the event of discomfort, significant hindrance, pain, variation in limb volume, abnormal sensations or change in colour of the extremities, remove the device and consult a healthcare professional.
For hygiene, security and performance reasons, do not re-use the product for another patient.
Do not use the device in case of application of certain products on the skin (creams, ointments, oils, gels, patches...).

Do not wear the product in a medical imaging machine.
Do not wear the product when driving a vehicle.
It is recommended to adequately tighten the device to achieve support/stabilisation without restricting blood circulation.

Undesirable side-effects

This device can cause skin reactions (redness, itching, burns, blisters, etc.) or wounds of various degrees of severity.
Possible risk of venous thrombosis.
Possible risk of ulcers.
Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

Instructions for use/Application

The fitting of the device must be done while seated.
The systematic use of a long sock is recommended when wearing the device.
The use of a closed shoe is recommended to strengthen the joint stabilisation.
Loosen the straps. Ⓐ
Place the ankle between the two lateral shells ensuring the shells are properly centered. Ⓑ
If necessary, adjust the width of the heel strap, using the self-fastening areas located on the lower part of the lateral shells. Ⓒ
Tighten the lower strap and fasten securely using the self-fastening closure. Ⓓ
Tighten the upper strap and fasten securely using the self-fastening closure. Ⓔ
If necessary, readjust the straps. Ⓕ
Loosen the shoe completely.
Carefully slide the foot with the device into the shoe.
Tighten the shoe. Ⓖ
If necessary, readjust the straps.

Care

Product can be washed in accordance with the instructions shown on this leaflet and on the label.
Shells: wipe with a damp cloth.
Foam padding: remove the foam pads from the shells before washing.
Replace in the original location before next use. Machine washable at 30°C (delicate programme). The device has been tested over 20 wash cycles. If possible use a washing net. Do not use detergents, fabric softeners or aggressive products (products containing chlorine). Do not dry clean. Do not tumble-dry.
Do not iron. Squeeze out excess water. Dry flat. Dry away from any direct heat source (radiator, sun, etc.).

Storage

Store at room temperature, preferably in the original packaging.

Disposal

Dispose of in accordance with local regulations.

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Ajustar el zapato. ©