THUASNE

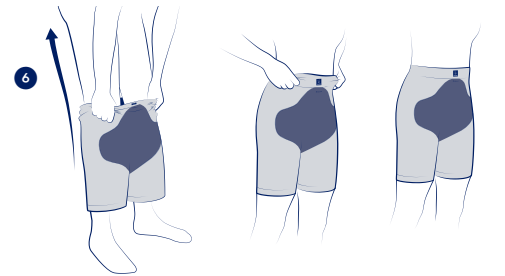
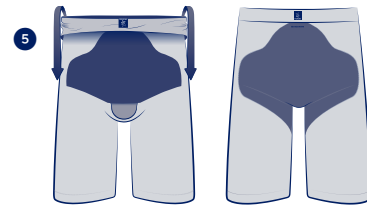
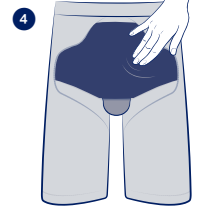
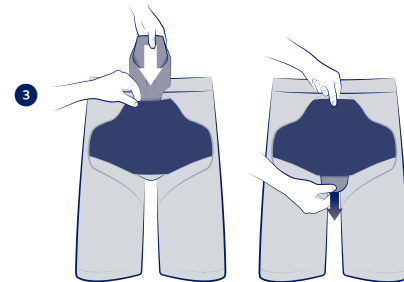
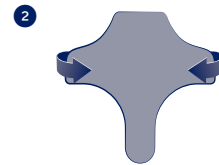
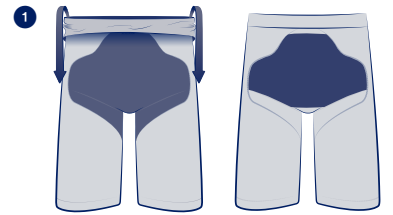
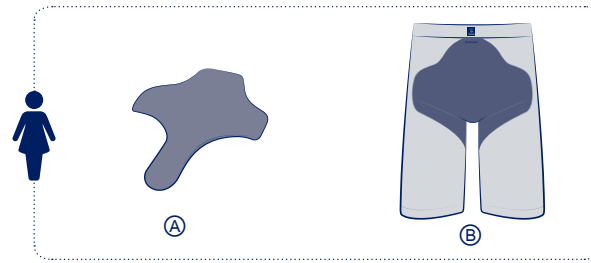
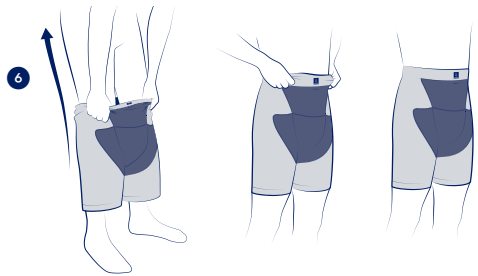
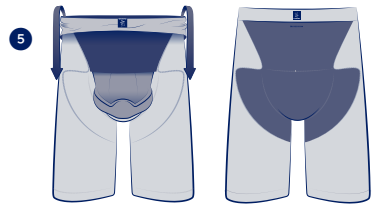
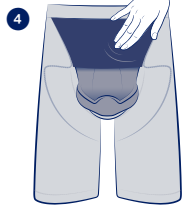
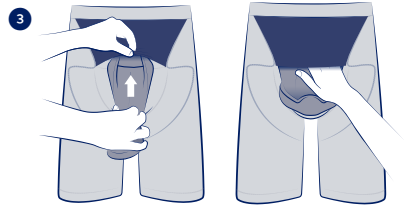
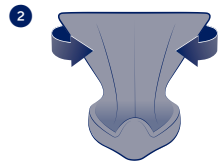
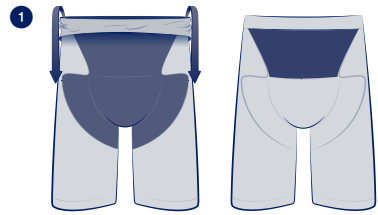
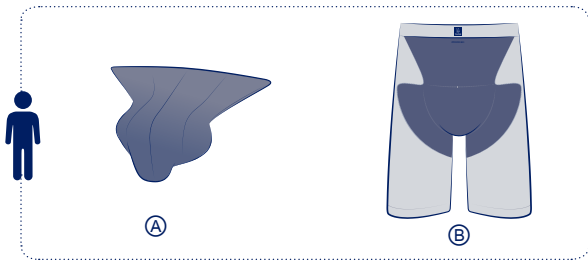
MOBIDERM Intimate

fr	Short de compression et de mobilisation	6
en	Mobilizing and compression shorts.....	7
de	Kompressions- und Mobilisierungs-Shorts	8
nl	Compressie- en mobilisatieshort	9
it	Dispositivo di compressione e di mobilitazione	11
es	Short de compresión y movilización	12
pt	Calções de compressão e de mobilização	13
da	Kompressions- og mobiliseringsshorts.....	14
fi	Kompressio- ja mobilisointishortsit	16
sv	Shorts som stimulerar och aktiverar	17
el	Σορτς συμπίεσης και κινητοποίησης.....	18
cs	Kompresní a mobilizační šortky.....	19
pl	Spodenki uciskowe i mobilizujące tkanki	21
lv	Kompresijas un mobilizācijas šorti.....	22
lt	Kompresiniai palaikomieji šortai	23
et	Surve- ja mobiliseerimiseseade.....	24
sl	Kompresijske in mobilizacijske kratke hlače	26
sk	Kompresívna a mobilizačná pomôcka.....	27
hu	Kompressziós és mobilizáló short	28
bg	Шорти за компресия и мобилизация	29
ro	Colanți scurți de compresie și de mobilizare.....	31
ru	Мобилизующие компрессионные шорты	32
hr	Kratke hlače za kompresiju i imobilizaciju	33
zh	具有压力和活血功效的短裤	35
ja	圧迫・弾性バンティ	35
ko	압박 및 모빌라이징 쇼트 팬츠	37
ar	شورت داخلي للضغط والتحرك	39

		cK (cm)	cG (cm)
	1	88 - 94	45 - 54
	2	95 - 101	48 - 58
	3	102 - 108	51 - 62
	4	109 - 115	54 - 66
	5	116 - 122	57 - 69
6	123 - 129	60 - 72	

	cK (cm)	cG (cm)
fr	Circonférence bassin	Circonférence haut de cuisse
en	Hip circumference	Upper thigh circumference
de	Beckenumfang	Umfang des oberen Oberschenkels
nl	Bekkenomtrek	Dijomtrek bovenaan
it	Circonferenza bacino	Circonferenza coscia nel punto più alto
es	Circonferencia de la pelvis	Circonferencia de la parte superior del muslo
pt	Circunferência da bacia	Circunferência da parte superior da coxa
da	Omkreds af bækken	Omkreds af den øverste del af låret
fi	Lantionympärys	Reidenympärys
sv	Omkrets runt höften.	Omkrets runt låret.
el	Περίμετρος λεκάνης	Περίμετρος άνω τμήματος μηρού
cs	Obvod boků	Obvod horní části stehna
pl	Obwód miednicy	Obwód górnej części uda
lv	Iegurņa apkārtmērs	Augšstilba apkārtmērs
lt	Klubų apimtis	Šlaunies viršutinės dalies apimtis
et	Vaagna ümbermõõt	Reie ümbermõõt
sl	Obseg medenice	Obseg zgornjega dela stegen
sk	Obvod panvy	Obvod vrchnej časti stehna
hu	Csípőbőség	Felső combbőség
bg	Обиколка на таза	Горна обиколка на бедрото
ro	Circumferința bazin	Circumferința coapsei
ru	Обхват таза	Обхват верхней части бедра
hr	Opseg zdjelice	Opseg gornjeg dijela bedara
zh	臀围	大腿围
ja	骨盤周囲	大腿上部周囲
ko	골반 둘레	허벅지 상단 둘레
ar	محيط الحوض	محيط أعلى الفخذ

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en

MOBILIZING AND COMPRESSION SHORTS

Description

The MOBIDERM Intimate range offers a variety of mobilizing and compression garments for subcutaneous or indurated edema.

These garments are underwears designed to treat pelvic and/or genital edema. Each garment comprises:

- compression shorts (polyamide, elastane, polyester) ⓐ,
 - a removable MOBIDERM genital mobilization pad in small blocks (polyester, polyamide, acrylic adhesive, polyethylene foam) ⓐ.
- MOBIDERM Intimate can be worn with or without the genital mobilization pad, depending on the patient's needs and the recommendations of the healthcare professional.

The garment comes in a different style for men and women, each available in six sizes.

The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table.

Mechanism of action

MOBIDERM Intimate mobilizing and compression garments provide:

- compression around the pelvis and buttocks, to encourage lymphatic drainage and help maintain or reduce swelling;
- subcutaneous drainage of edematous and indurated areas, by creating a pressure gradient between the parts of the skin compressed by the foam blocks of the MOBIDERM genital pad and the surrounding areas, which helps drain away the excess lymph fluid and reduce swelling.

The garments are designed to be worn day and night.

They provide graduated pressure (between the waist, the hips and the thighs):

- 15-20mmHg around the pelvis

- 5-10mmHg on the abdomen and mid thighs.

Indications

MOBIDERM Intimate mobilizing and compression devices are indicated in the treatment of pelvic and genital lymphedema.

Instructions for use

-  We recommend washing your new garment before use.
- Always make sure that the product is intact before each use.
- MOBIDERM Intimate should be worn against the skin, like an undergarment.
- The genital mobilization pads are reversible.

Inserting the male genital mobilization pad :

- The genital mobilization pad should be inserted before putting on the shorts.
- Turn the shorts inside out and locate the inner pocket over the groin. ❶
- Place the genital pad face up with the concave section towards the bottom, and fold the corners into the middle. ❷
- Insert the genital pad into the pocket, placing the top in first then sliding the pad upwards. ❸
- Unfold and adjust the genital pad, making sure it fits neatly into the pocket. ❹
- Turn the shorts right-side out. ❺

Inserting the female genital mobilization pad :

- The genital mobilization pad should be inserted before putting on the shorts.
- Turn the shorts inside out and locate the inner pocket over the groin. ❶
- Place the genital pad flat with the tab towards the bottom, and fold the wings into the middle. ❷
- Insert the genital pad into the pocket at the waist, placing the narrowest part in first then sliding the pad down until the tab is sticking out of the other side of the pocket. ❸
- Unfold and adjust the genital pad, making sure it fits neatly into the pocket. ❹
- Turn the shorts right-side out. ❺

Putting on the shorts :

- Place both legs in the shorts, then pull the garment gently and evenly up towards the waist. ❶
- Men: When pulling up the panty, we recommend holding the genital pad by the tips of the triangle in order to keep it in place.
- Avoid tugging on the fabric or waistband more than necessary.
- The pocket containing the genital pad should sit at the front.
- Adjust the garment to make sure there are no folds, do not roll up the bottom of the garment.

Contraindications

Skin infections on the pelvis or legs.

Pregnant women (from the first month).

Known allergies to any of the components.

Lower limb peripheral arterial disease (PAD) with ankle brachial index (ABI) < 0.6. Decompensated heart failure.

Septic thrombosis of the proximal veins.

Phlegmasia coerulea dolens (painful blue edema with arterial compression) in the thighs.

Posthectomy (circumcision) within the past three months.

Female patient with marked localized edema of the labia minora only.

Precautions

Strictly comply with your healthcare professional's prescription and recommendations for use.

Do not wear the garment over damaged skin.

Do not let the garment contact the vaginal mucosa (e.g. edema of the labia minora). If you experience significant discomfort, skin damage or infection, or unusual sensations, remove the garment and consult a health care professional.

If the edema worsens, consult a health care professional.

Due to the constantly changing nature of this condition, we recommend regular check-ups with a health care professional to ensure the device is still suitable for you.

Do not use when swimming.

Do not apply any creams before putting on the shorts as they may damage the fabric.

Take care not to damage the device with your nails when fitting it.

If you suffer from advanced venous disease, venous ulcer or lymphedema, consult a health care professional for advice on a suitable full-leg compression garment.

If using with compression stockings, we recommend wearing MOBIDERM Intimate over the stockings.

The genital mobilization pad is designed exclusively for use with MOBIDERM Intimate. The two items should be used together.

For reasons of hygiene, safety and performance, do not share your garment with anyone else. The product should not be reused for another patient.

Undesirable side-effects

This device can cause skin reactions (redness, itching, burns, blisters, etc.) or wounds of various degrees of severity.

Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

Care

To ensure your compression garment lasts as long as possible and to prevent infection, it is important to wash it every day, or after each use. MOBIDERM Intimate mobilizing and compression garments are designed for daily use over six months, or for use on alternate days over a year i.e. 180 washes. After this time, garments should be replaced to ensure the treatment is effective and technical performance is not affected.

Machine washing:

Wash the shorts and genital pad on a normal cycle at 40°C. Use a laundry detergent without fabric softener. Do not use chlorine-based detergents or harsh products. Remove the genital mobilization pad from the shorts. Place the shorts and genital pad in a mesh laundry bag to prevent damage by other garments (e.g. snagging on touch-fastening tape or zippers). Air dry, away from direct heat (e.g. radiator, sun, etc.). Do not tumble-dry. Do not iron.

Disposal

Check local regulations for how to dispose of MOBIDERM Intimate mobilizing and compression garments.

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©Thuasne - 2042501 (2021-08)
THUASNE UK Ltd