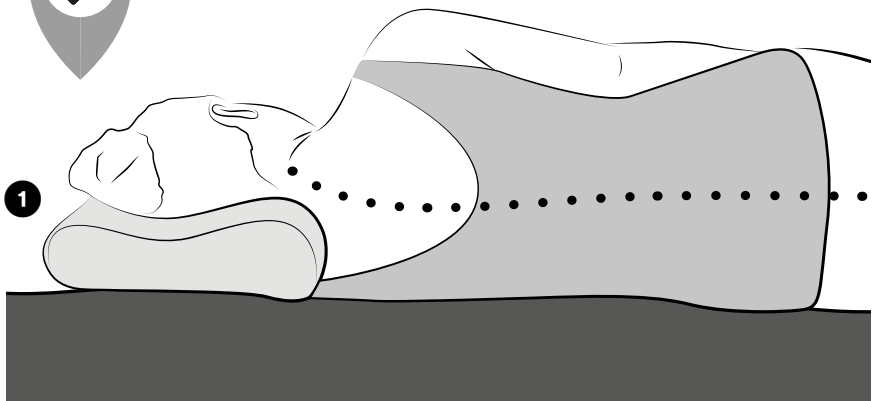
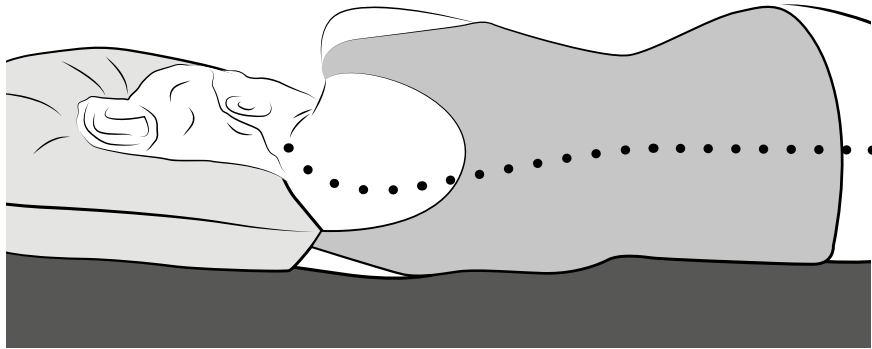
**THUASNE**

## **Cervi+ / Cervi+ Maxi**

W0460

fr	Oreiller cervical en mousse à mémoire de forme .....	3
en	Memory foam cervical pillow .....	4
de	Nackenkissen aus Memory-Schaumstoff .....	4
nl	Neksteunkussen in geheugenschuim .....	5
it	Guancia cervicale in memory foam .....	6
es	Almohada cervical de espuma con memoria de forma .....	7
cs	Polštář na krční páteř z pěny s tvarovou pamětí .....	8
pl	Poduszka profilowana z pianki z pamięcią kształtu .....	8
sk	Vankúš s tvarovou pamäťou .....	9
hu	Memóriahabból készült nyaki párna .....	10
ar	وسادة نوم مزودة بأسفنج ذي ذاكرة الشكل .....	11

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## MEMORY FOAM CERVICAL PILLOW

### Description/Intended users

Ergonomic memory foam pillow for adults sleeping on their side or back.

Available in 2 sizes.

### Composition

Viscoelastic foam: 100% polyurethane  
Stretch terrycloth cover: 80% cotton, 20% polyester  
Inner cover: 100% polyester

### Properties/Mode of action

Memory foam pillow that supports the cervical vertebrae and adapts to body shape and changes of position during sleep.

The foam provides the right level of support to relieve the neck of the head's weight and help the vertebrae align naturally. ①

The design of the pillow, with two waves of different heights, stabilizes the cervical vertebrae and safeguards the head's movements.

Permanent contact between the neck and the pillow aids relaxation.

### Indications

Back pain, neck pain (including post-traumatic).  
Cervical arthritis.

### Contraindications

Do not use in the event of known allergy to any of the components.

Do not apply the product in direct contact with broken skin.

### Precautions

#### Important warnings

Verify the product's integrity before every use.

Do not use the device if it is damaged.

The health care professional who supplied the product must make sure that it is appropriate for the patient.

Follow the advice of the health care professional who prescribed or supplied the product and consult them in case of doubt.

For hygiene and performance reasons, do not re-use the product for another patient.

Never allow children to play with the product.

Do not compress the pillow excessively or for long periods, as this could compromise the properties of the foam.

#### Undesirable side-effects

This pillow can cause skin reactions.

Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

#### Instructions for use

The pillow has two waves of different heights: choose whichever is more comfortable, depending on whether you sleep on your side or your back.

An adjustment period of a few days may be necessary.

#### Care

The cover is machine-washable at 40 °C. The care instructions are on the label.

Do not wash the foam.

#### Storage

Store at room temperature, in a dry, ventilated place.

#### Disposal

Dispose of in accordance with local regulations.

#### Technical characteristics

Fire resistance: product tested to the ISO 12952-1:2010 standard.

Density: 73 kg/m<sup>3</sup>.

Dimensions (L x W x H):

- Cervi+ pillow: 50 x 32 x 10,5/8 cm.
- Cervi+ Maxi pillow: 65 x 36 x 11,5/10 cm.

Replacement covers for Cervi+ and Cervi+ Maxi pillows are available from Thuasne dealers.

#### Commercial warranty agreement and warranty limitations

Thuasne provides a free commercial three-year warranty to users living in the territory where the product was purchased against faults and manufacturing defects leading to sagging of the memory foam.

The limited warranty is effective from the date of purchase of the product by the end-user.

The limited commercial warranty does not apply to any defects in manufacturing and workmanship in case of:

- misuse of the product or any damage caused outside the normal conditions of use and storage of the product as described in the instructions for use,
- excessive or prolonged compression of the product,
- damage that occurs due to attempts to modify the product.

Faults and manufacturing defects in the removable cover are excluded from this commercial warranty.

Any claim under this commercial warranty must be sent by the user to the entity where the product was purchased, which will forward this claim to the corresponding Thuasne entity.

Any warranty claim will first be reviewed by Thuasne to determine if the conditions of the limited warranty are fulfilled and if the claim does not fall into one of the cases of exclusion of the commercial warranty.

To benefit from the warranty, the buyer must mandatorily provide an original and dated proof of purchase of the product.

If the conditions of the limited warranty are fulfilled and the claim is made by the user or its legal representative (parents, guardian...) within the warranty delays indicated above, the buyer will get a new substitution product.

It is expressly agreed that this commercial warranty is in addition to the legal warranties binding the entity which sold the product to the user, in accordance with the applicable local legislation in the country of purchase of the product.

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©Thuasne - 2032402 (2022-01)
Thuasne UK Ltd