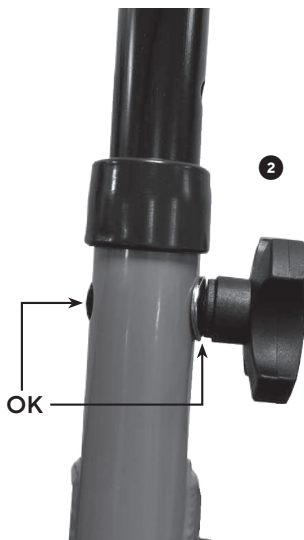
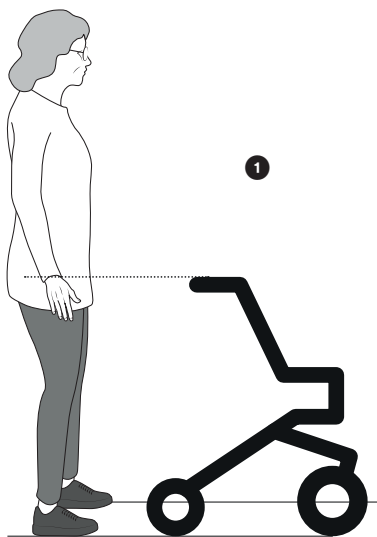
THUASNE

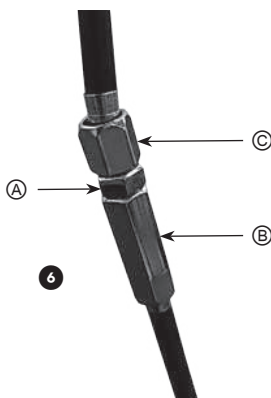
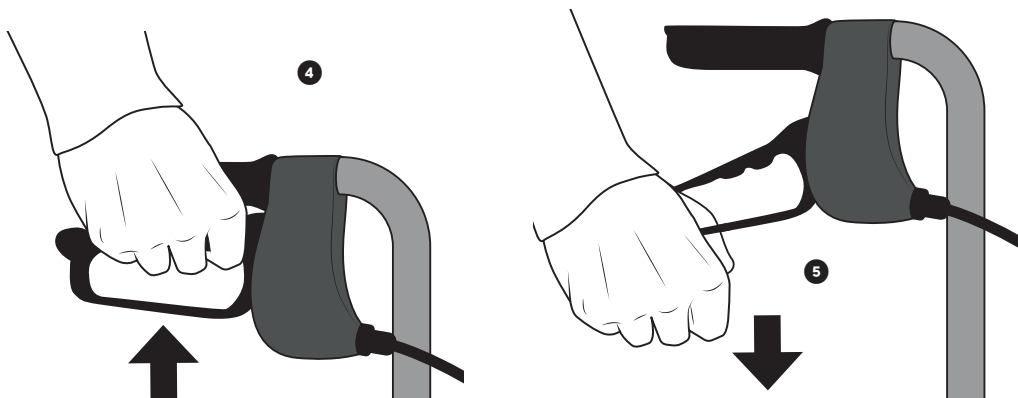
Move Light

V05041

fr	Rollateur	4
en	Rollator	6
de	Rollator	8
nl	Rollator	10
it	Rollator	12
es	Andador	14
cs	Chodítka	16
pl	Podpórka	18
sk	Štvorkolka	20

MD

fr	Dispositif médical
en	Medical device
de	Medizinprodukt
nl	Medisch hulpmiddel
it	Dispositivo medico
es	Producto sanitario
cs	Zdravotnický prostředek
pl	Wyrób medyczny
sk	Zdravotnícka pomôcka

fr	Lire attentivement la notice
en	Read the instruction leaflet carefully
de	Lesen Sie die Gebrauchsanweisung sorgfältig durch
nl	Lees voor gebruik de gebruiksaanwijzing aandachtig door
it	Leggere attentamente le istruzioni
es	Leer atentamente las instrucciones
cs	Pozorně si přečtěte návod
pl	Należy dokładnie przeczytać instrukcję
sk	Tento návod si pozorne prečítajte

ROLLATEUR MOVE LIGHT

Description/Destination

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Description/Destination

4-wheel rollator.

This device helps temporarily or permanently mobility impaired individuals to maintain their freedom of movement by providing them with an additional point of support. It also has a seat with a backrest for taking short rests. It can be used inside and outside.

Main components

Frame: aluminium

Handles: PVC

Backrest (cover): thermoplastic

Seat (cover): PVC

Wheels/tires: thermoplastic

Basket: plastified steel

Indications

Post-operative and post-traumatic support.

Muscle weakness, neuromuscular disorders, stability problems, impaired balance, arthritis of the lower limbs.

Degenerative conditions of the lower limbs.

Contraindications

Not suitable for patients weighing > 100 kg.

Do not use in the event of known allergy to any of the components.

Do not use in the event of severe perceptual function deficit or severe balance disorders.

Precautions

Important warnings

Verify the product's integrity before every use.

Do not use the device if it is damaged.

Before using this product, check that:

- it is stable and rigid,
- the brakes are working properly,
- the wheels are in good condition and turn freely,
- no components are incorrectly attached or not properly screwed in place.

The health care professional who supplied the product must make sure that it is appropriate for the patient.

It is recommended that a health care professional assembles and adjusts the device and supervises the first use. In particular, it is essential that they check and adjust the tension of the brake cables.

Follow the advice of the health care professional who prescribed or supplied the product and consult them in case of doubt.

Do not use the device for any use other than its intended use: risk of injury or damage to equipment.

Never allow children to play with the product.

Beware of the risks of pinching when handling the device.

Always completely unfold the rollator before use.

Do not make any mechanical changes to the device, this could compromise its safety. Only original spare parts and accessories have to be used.

Take care when using the device on slippery surfaces (wet, snow-covered or icy floors or wet tiles, etc.), uneven, sloping or unstable surfaces (gravel, sand, leaves, etc.).

Do not use on stairs or escalators.

Do not leave exposed to direct sunlight for a long period of time. Check the temperature of the device surfaces, particularly before sitting down on the seat: risk of burns.

Undesirable side-effect

- Risk of discomfort in the event of incorrect adjustment of the device.
- Danger of falling or loss of balance, which can lead to serious injuries.

Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

Instructions for use

The rollator is supplied with a basket and a cane holder.

- ① Brake
- ② Basket
- ③ Strap
- ④ Cane holder
- ⑤ Rear wheel with brake
- ⑥ Pivoting front wheel
- ⑦ Crossbar
- ⑧ Seat
- ⑨ Backrest

Assembly/unfolding

1. Opening the rollator

Place the rollator on its wheels, push the crossbar downwards to completely open the rollator and lock.

2. Assembling the handles/adjusting the height

The height of the device must be adjusted with reference to the height of the patient. Incorrect adjustment can impair performance and place too much strain on the user.

The top of the handle must be approximately at the height of the wrist fold when the user's arms are alongside the body, wearing usual shoes. **1**

- Unscrew the knob located below the handle, remove the bolt and the washer. Insert the handle into the tube at the desired height.

- Then put the bolt back into the hexagonal opening, put the washer in place and tighten the knob by hand, making sure the bolt is fully pushed in. **2**

Make sure that the handles are fixed at the same height on both sides.

3. Installing the backrest

Press the push-buttons while simultaneously sliding the right and left tubes into the frame supports. Check that the push-buttons are properly engaged before use.

4. Installing the accessories

Basket: lift up the seat, place the basket on the crossbar and put the seat back down.

Cane holder: pivot the upper and lower parts of the cane holder outwards.

Folding

1. To fold away the rollator, release the brakes, lift up the seat, remove the basket and pull the strap to raise the bar upwards.

2. Then lock the brakes so that the rollator cannot move. Once folded, the rollator stands on its own and is easy to transport. ③

Braking system

• Quick braking

To brake, hold on to the handles and squeeze the brake levers. ④

• Parking brakes

To immobilize the rollator (e.g., before sitting down), carefully push the brake levers down until they lock (click). ⑤

To start moving again, gently squeeze the levers to release the brakes.

• Adjusting the brakes

- If braking is not sufficient, loosen the locknut (A), holding the support (B), then turn the adjustment screw (C) anticlockwise. ⑥

- If braking is too hard, proceed as above but turn the screw clockwise. ⑥

Remember to tighten the locknut again after adjusting to the optimal setting.

If necessary, adjust the setting at the screws on the front of the handles.

⚠ Important warnings

Before using the seat:

- check that the rollator is on a flat, stable surface,
- it is essential for the brakes to be in the "parking brakes" position.

Do not sit at an angle or lean to the side or back.

The rollator seat is only intended for taking short rests.

Do not attach handbags, carrier bags or any objects other than those supplied with the device: risk of tipping.

The device must not be used as an aid to get up from the sitting position: risk of unbalancing.

Maintenance and care

Maintenance

If any part of the device becomes detached, loose or visibly damaged, contact a specialized dealer or health care professional.

If necessary, replacement parts and original accessories are available from your Thuasne product dealer.

Care

Clean with a mild cleaning agent and a soft cloth.

Do not clean the handles or the wheels with greasy substances: risk of slipping.

If necessary, use a mild contact disinfectant or 70° alcohol. In an institution, follow the recommendations of the hygiene manager.

Storage

Store at room temperature, in a dry, ventilated place.

Disposal

Dispose of in accordance with local regulations.

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©Thuasne - 2026601 (2021-01)