THUASNE®

Stomex

fr	 Ceinture de soutien abdominal pour stomie	5
en	 Abdominal support belt for stoma	6
de	 Bauchstützgurt für Stoma	7
nl	 Buikgordel voor ondersteuning van stoma	8
it	 Cintura di sostegno addominale per stomizzati	9
es	 Faja de sujeción abdominal para estoma	10
pt	 Cinta de suporte abdominal para ostomia	11
da	 Mavestøttebælte til stomi	12
fi	 Tukivyö vatsa-avanteelle	13
sv	 Stödjande bukbalte för stomi	14
el	 Ζώνη υποστήριξης κοιλιακής χώρας για κολοστομία	15
cs	 Stomický podprůrný břišní pás	17
pl	 Pas do stomii podtrzymujący mięśnie brzucha	18
lv	 Vēdera daļas atbalsta josta stomas pacientiem	19
lt	 Palaikomasis pilvo diržas esant ostomijai	20
et	 Abdominaalne stoomi tugivöö	21
sl	 Oporni pas za trebušni predel in stomo	22
sk	 Brušný podporný pás pre stomických pacientov	23
hu	 Haskötő sztómához	24
bg	 Поддържащ колан за коремната област при stomатит	25
ro	 Centură de susținere abdominală pentru stomă	26
ru	 Опорный брюшной корсет для стомы	27
hr	 Pojas za abdominalnu potporu stome	29
zh	 用于气孔的腹部支撑带	30
ar	 حزام دعم البطن عند إجراء الفغر	31

Instructions d'utilisation - Instructions for use - Gebrauchsanweisung - Gebruiksaanwijzingen - Istruzioni per l'uso - Instrucciones de uso - Instruções de utilização - Anvisninger vedrørende brug - Käyttöohjeet - Råd vid användning - Οδηγίες χρήσης - Pokyny k používání - Szczegóły użytkowania - Lietošanas instrukcija - Naudojimo instrukcijos - Kasutusjuhised - Navodila za uporabo - Pokyny na používanie - Használati utasítás - Инструкции за употреба - Instrucțiuni de utilizare - Инструкция по эксплуатации - Upute za uporabu - 使用指南 - إرشادات الاستعمال

	1	60 - 80 cm
	2	80 - 100 cm
	3	100 - 130 cm
	4	130 - 160 cm

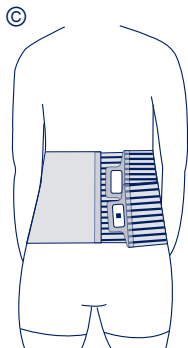
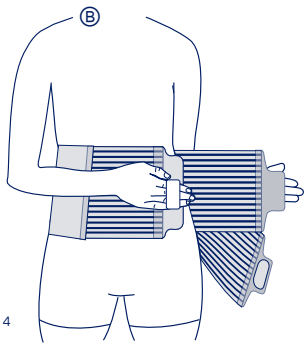
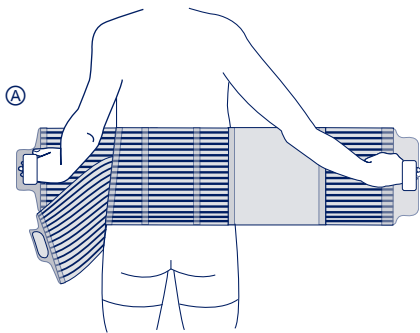
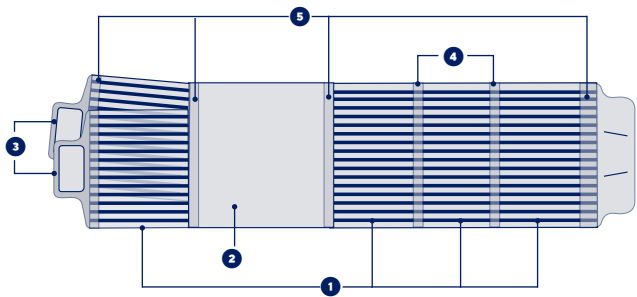
fr	Tour de taille	Hauteur : 25 cm
en	Waist circumference	Height: 25 cm
de	Taillenumfang	Höhe: 25 cm
nl	Tailleomvang	Hoogte: 25 cm
it	Circonferenza vita	Altezza: 25 cm
es	Contorno de cintura	Altura: 25 cm
pt	Perímetro de cintura	Altura: 25 cm
da	Taljemål	Højde: 25 cm
fi	Vyötärönympärys	Korkeus: 25 cm
sv	Midjemått	Höjd: 25 cm
el	Περίμετρος μέσης	Ύψος: 25 εκ
cs	Obvod pasu	Výška: 25 cm
pl	Obwód w talii	Wysokość: 25 cm
lv	Vidukļa apkārtmērs	Augstums: 25 cm
lt	Juosmens apimtis	Aukštis: 25 cm
et	Vööümbmõõt	Kõrgus: 25 cm
sl	Obseg pasu	Višina: 25 cm
sk	Obvod pásu	Výška: 25 cm
hu	Derékbőség	Magasság: 25 cm
bg	Обиколка на талията	Височина: 25 cm
ro	Circumferința taliei	Înălțime: 25 cm
ru	Обхват талии	Высота: 25 cm
hr	Opseg struka	Visina: 25 cm
zh	腰圍	高度 : 25 厘米
ar	محيط الخصر	الارتفاع: 25 سم

fr	Compression	Soutien
en	Compression	Support
de	Kompression	Stütze
nl	Compressie	Ondersteuning
it	Compressione	Sostegno
es	Compresión	Sujeción
pt	Compressão	Suporte
da	Kompression	Støtte
fi	Kompressio	Tuki
sv	Kompression	Stöd
el	Συμπίεση	Υποστήριξη
cs	Komprese	Podpora
pl	Kompresja	Podtrzymywanie
lv	Kompresija	Atbalsts
lt	Kompresija	Palaikymas
et	Kompressioon	Toetus
sl	Kompresija	Podpora
sk	Kompresia	Podpora
hu	Szorítás	Megtámasztás
bg	Компресия	Поддръжка
ro	Compresie	Susținere
ru	Компрессия	Поддержка
hr	Kompresija	Potpora
zh	压紧	支撑
ar	الضغط	الدعم

fr

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en ABDOMINAL SUPPORT BELT FOR STOMA

Description/Destination

The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table*.

Composition

Fastening "loop knit" elastic fabric. ❶
Abdominal section made of non-run elastic fabric - "clean-cut" for cutting openings to insert a stoma bag. ❷
Self-fastening with hand-loop. ❸
Rigid, conformable and removable dorsal stays. ❹
Flexible and removable abdominal stays. ❺
Textile components: polyurethane - elastane - polyamide - polyester - cotton - polypropylene
Non-textile components: brushed tempered steel - polyoxymethylene - polyamide

Properties/Mode of action

The device ensures strong support and adaptable compression to the abdomen thanks to the elastic compact fabric.

Indications

Stoma.
Post-operative abdominal conditions.

Contraindications

Do not apply the product in direct contact with broken skin.
Do not use in the event of known allergy to any of the components.
Do not use on pregnant women.
Do not use in the event of a hiatus hernia.
Do not use in the event of bone cancer with spinal metastases.
Do not use in the event of circulatory, pulmonary or cardiovascular problems in patients for whom an increase in arterial blood pressure is not recommended.

Precautions

Verify the product's integrity before every use.
Do not use the device if it is damaged.

Choose the appropriate size to fit the patient, referring to the size chart.

Strictly comply with your healthcare professional's prescription and recommendations for use.

It is recommended that a healthcare professional adjust the stays to the shape of the patient's back before the belt is worn for the first time.

It is recommended that a healthcare professional supervises the first application.

Consult a healthcare professional before use in the event of a weak pelvic floor.

Do not wear the device in a medical imaging machine.

In the event of discomfort, significant hindrance, pain, abnormal sensations, remove the device and consult a healthcare professional.

This product is intended for the treatment of a given condition. Its duration of use is limited to this treatment only.

For hygiene and performance reasons, do not re-use the product for another patient.

It is recommended to adequately tighten the device to achieve support without excessive compression.

Do not use the device in case of application of certain products on the skin (creams, ointments, oils, gels, patches...).

The use of a medical device by a child, should be done under the supervision of an adult or a healthcare professional.

Undesirable side-effect

This device can cause skin reactions (redness, itching, burns, blisters, etc.) or wounds of various degrees of severity.

Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

Instructions for use/Application

Follow the healthcare professional's recommendations concerning cutting an opening for the stoma bag.

Position the belt around the waist, using the hand-loop. ❶

The belt must be centred on the spine: the dorsal stays should be positioned on either side of the spine.

The bottom of the belt must stop at the top of the crease between the buttocks.

Hold the end without a hook on the abdomen. ❷

Hold the other end and place the hook on the fastening "loop knit" elastic fabric. Adjust to the desired level of tightness. ❸

The level of compression can be adjusted as required throughout the day.

Care/Maintenance

Close the self-fastening tabs before washing. Machine washable at 60°C. Do not machine wash the device more than 100 times. Do not dry clean. Do not use detergents, fabric softeners

or aggressive products (products containing chlorine). Do not tumble-dry. Do not iron. Squeeze out excess water. Dry away from any direct heat source (radiator, sun, etc.). Dry flat.

Storage

Store at room temperature, preferably in the original packaging.

Disposal

Dispose of in accordance with local regulations.

* Not applicable to made-to-measure medical devices, where measurements will be taken individually for each patient.

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Manter a segunda extremidade e colocar o gancho sobre o tecido elástico de "presilha" fixante. Ajustar o nível de aperto pretendido. © Durante o dia, é possível ajustar o nível de compressão em função das necessidades.
THUASNE UK Ltd**
Thuasne SAS - SIREN/RCS Nanterre 542 091 186